

<b>Manufacturer name:</b>	Baxter Healthcare
<b>About the Recall:</b>	<p>Baxter has received reports relating to the new battery charger dongle which was implemented in July 2023. Baxter has determined that the Life2000 Ventilator systems were either failing to charge or had intermittent charging behavior due to damage to the battery charger dongle. Damage of the battery charger dongle prevents the ventilator’s internal battery from charging.</p>
<b>Products(s) numbers/models that are affected:</b>	<p>The Urgent Medical Device Recall applies to the Life2000 ventilator with product code MS01-0118 with an attached battery charger dongle. The impacted ventilators were distributed in the United States between Aug. 21, 2023 and April 2, 2024</p>
<b>URL Link(s) to Manufacturer Notice/Web Site:</b>	<p><a href="#">Baxter Issues Urgent Medical Device Recall for Life2000 Ventilator Due to Potential Battery Charger Dongle Damage   Baxter</a></p>
<b>Actions to be taken by customer/Patient:</b>	<p>Patients/Caregivers should inspect the condition of the battery charger dongle and confirm the device is charging properly. If damage is observed or the battery is not charging properly, patients should contact Baxter Home Care Customer Service to have the device replaced immediately. If no damage is observed and the battery is charging properly, patients may continue to use the ventilator. For further information regarding the recall, you may contact Baxter Acute Care Customer Service at 800-426- 4224 option 2.</p>