

<b>Manufacturer name:</b>	Baxter Healthcare								
<b>About the Recall:</b>	<p>Baxter has received reports relating to the new battery charger dongle which was implemented in July 2023. Baxter has determined that the Life2000 Ventilator systems were either failing to charge or had intermittent charging behavior due to damage to the battery charger dongle. Damage of the battery charger dongle prevents the ventilator's internal battery from charging.</p>								
<b>Products(s) numbers/models that are affected:</b>	<table border="1" data-bbox="386 856 1528 953"> <thead> <tr> <th>Product Code</th> <th>Product Description</th> <th>Impacted Devices</th> <th>Device Identifier Number</th> </tr> </thead> <tbody> <tr> <td>MS-01-0118*</td> <td>Life2000 Ventilator</td> <td>Refer to Attachment A</td> <td>00887761978089 or 00815410020537</td> </tr> </tbody> </table> <p>*Product code MS-01-0118 appears on the ventilator. Product codes BT-20-0002, BT-20-0002A, and BT-20-0002AP appear on the shipping cartons.</p>	Product Code	Product Description	Impacted Devices	Device Identifier Number	MS-01-0118*	Life2000 Ventilator	Refer to Attachment A	00887761978089 or 00815410020537
Product Code	Product Description	Impacted Devices	Device Identifier Number						
MS-01-0118*	Life2000 Ventilator	Refer to Attachment A	00887761978089 or 00815410020537						
<b>URL Link(s) to Manufacturer Notice/Web Site:</b>	<a href="http://www.baxter.com">www.baxter.com</a>								
<b>Actions to be taken by customer/Patient:</b>	<p>Patients and/or caregivers should check the battery charger dongle regularly to ensure no damage has occurred to the dongle. You should always have a back up of ventilation or oxygen therapy available. Should you find that the battery charger dongle has damage or not charging properly contact Baxter Acute Care customer service.</p> <p>For further information regarding the recall, you may contact Baxter Acute Care Customer Service at 800-426- 4224 option 2.</p>								